**INVACARE® COMMODES
CHAISE D'AISANCE INVACARE®
ORINALES INVACARE®**

**Refer to page 2.
Se référer à la page 7.
Refiérase a la página 13.**

All Invacare commodes below offer a Lifetime Limited Warranty.

ALL-IN-ONE COMMODE

Invacare's most versatile commode can be used inside the bathroom as a toilet safety frame or raised toilet seat or outside the bathroom as a stationary commode.

- Splash shield and pail with lid
- Removable backrest with wing nuts
- Weight limit: 300 lbs.

Model 9630-1/9630-4 - Gray with arms

Model 9650-4 - Aluminum with arms

SAFEGUARD COMMODE

Designed to be used inside the bathroom as a toilet safety frame or raised toilet seat.

- Splash shield and pail with lid
- Weight Limit: 300 lbs.

Model 9610-4 - Gray with Arms

Width				Depth			Height		
Base	Between Arms	Seat Overall	Seat Inside	Base Overall	Seat Overall	Seat Inside	Overall Min./Max.	Seat Min./Max.	Seat-to-Arm
25.5"	19"	16"	8.25"	19.25"	14"	10"	23.75"/29.75"	16"/22"	7.5"

Above specifications are the same for model nos. 9610, 9630, and 9650.

EXTRA-WIDE COMMODE

A heavy-duty bedside commode offers extra width between arms and a greater weight capacity.

- Rectangular pan and pan holder
- Weight Limit: 450 lbs.

Model 6496 - Gray with Arms

Width				Depth			Height		
Base	Between Arms	Seat Overall	Seat Inside	Base Overall	Seat Overall	Seat Inside	Overall Min./Max.	Seat Min./Max.	Seat-to-Arm
27"	23.5"	16"	8.25"	19.25"	16"	12.5"	27.25"/31.25"	18.75"/22.75"	8.5"

Specifications are subject to change without notification.

SAVE THESE INSTRUCTIONS

NOTE: Check all parts for shipping damage. In case of shipping damage, DO NOT use. Contact Dealer/Carrier for further instruction.

SAFETY SUMMARY

To ensure the proper use of Invacare Commodes, these instructions **MUST** be followed:

GENERAL WARNINGS

DO NOT install or use this equipment without first reading and understanding these instructions. If you are unable to understand the **Warnings, Cautions or Instructions**, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment - otherwise, injury or damage may occur.

Users with limited physical capabilities should be supervised or assisted when using the commode.

Leg extensions with rubber tips **MUST** be in contact with the floor at all times.

The Commode Model Nos. 9610, 9630, and 9650 have a weight limitation of 300 lbs. (135 kg.). The Commode Model No. 6496 has a weight limitation of 450 lbs. (204 kg.).

INSTALLATION WARNINGS

Ensure that the snap buttons fully protrude through the same respective adjustment hole of each leg extension. This ensures that the leg extensions are securely locked in position and an even height adjustment is achieved.

Make certain **ALL** screws, nuts and/or bolts are tight at all times.

Inspect rubber tips on the leg extensions for rips, tears, cracks or wear or if they are missing. Replace them immediately if any of these conditions exist.

INSTALLATION WARNINGS CONTINUED

Models - 9650 and 9630 - Back tube wing nuts **MUST** be inspected regularly for proper tightness - otherwise injury or damage may occur.

LEG HEIGHT ADJUSTMENT (FIGURE 1)

NOTE: Refer to the GENERAL WARNINGS and INSTALLATION WARNINGS in the SAFETY SUMMARY of this instruction booklet.

1. Select desired height for commode by aligning snap button of each leg extension with appropriate adjustment hole in leg frame.
2. Ensure snap buttons are fully engaged into adjustment hole of each leg frame and are at the same height before using.

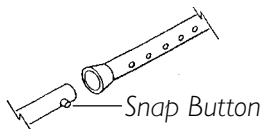
FIGURE 1 - LEG HEIGHT ADJUSTMENT INSTALLING/REMOVING THE COMMODE SEAT AND LID (FIGURE 2)

NOTE: Refer to the GENERAL WARNINGS and INSTALLATION WARNINGS in the SAFETY SUMMARY of this instruction booklet.

CAUTION

Before removing/installing seat and lid, allow seat/lid to reach room temperature if exposed to cold. This will help prevent the seat clamps from breaking when being removed/installed onto commode back frame rail.

1. Align the seat clamps of the commode seat onto the back frame rail. **(Each clamp should sit outside the pail holders)**. Push down on seat until it clamps firmly into place.
2. Repeat the same process for the lid.
3. When removing the seat or lid, remove the lid first and then the seat. To remove, grasp the back corner of the lid and/or seat and lift up firmly.

WARNING

Commode seat MUST be DOWN before sitting on the commode. Pail holders are NOT constructed to support the weight of an individual, ONLY the pail and its contents.

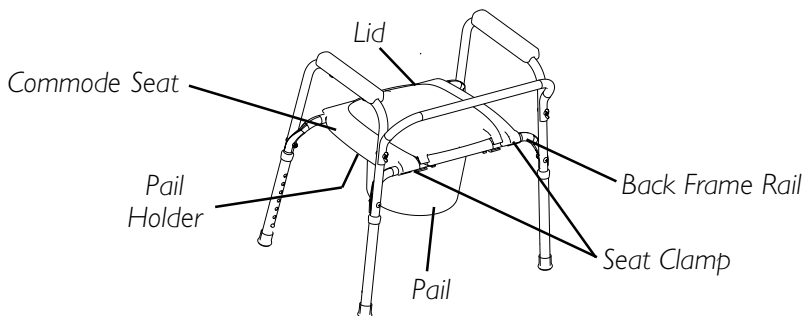
FIGURE 2 - INSTALLING/REMOVING THE COMMODOE SEAT AND LID

CARE AND LUBRICATION

CARE

Use a damp cloth and soap to clean.

LUBRICATION

To make raising or lowering the seat/lid easier, you may lubricate the commode back frame rail with a very small amount of petroleum jelly.

LIFETIME LIMITED WARRANTY

Invacare warrants this product to be free from defects in materials and workmanship.

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants its product to be free from defects in material and workmanship for the lifetime of use by original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back cover. Provide dealer's name, address, model number, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will use a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. **DO NOT** return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS, PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN AUTHORIZED INVACARE DEALER). SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER WARRANTIES WHATSOEVER, WHETHER EXPRESSED OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

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© 2003 Invacare Corporation
Invacare is a registered trademark of
Yes, you can. is a trademark of Invacare