**⚠ WARNING / AVERTISSEMENT / ADVERTENCIA :**

U.S. Federal law restricts this device to sale by or on the order of a physician. Read all warnings and instructions before use. Improper use can result in serious or fatal illness or injury. This product contains dry natural rubber.

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ILLUSTRATIONS






Before reading these Instructions For Use, open this page to view the illustrations. When reading Section IV on Set-Up, the illustrations on the left inside flap will match the illustrations in the text. These illustrations will be helpful to you as they are referenced throughout these Instructions For Use.

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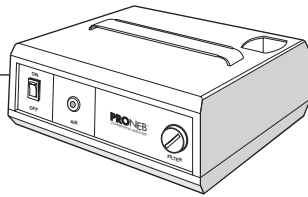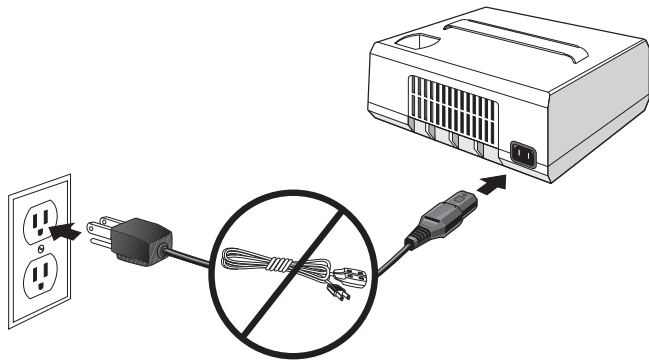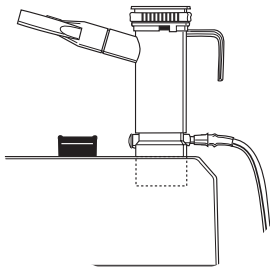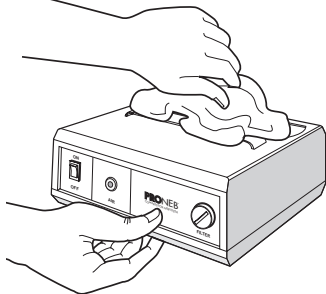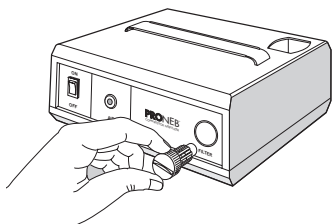A**B****C****D****E****F****G**

ENGLISH

I. FEATURES, FUNCTIONS AND SPECIFICATIONS

- | | |
|-----------------------|----------------------------|
| (1) PRONEB Compressor | (5) Nebulizer Holder |
| (2) On/Off Switch | (6) Power Cord Socket |
| (3) Air Outlet | (7) Three Prong Power Cord |
| (4) Air Filter | (8) Nebulizer Stand* |

SPECIFICATIONS OF COMPRESSOR:

Weight:	4.2 lbs. (1.9 Kg)
Dimensions:	3.75" x 6.5" x 8.25" (9.5 x 16.5 x 20.9cm)
Electrical Requirements:	120V AC, 60Hz, 1.3 A or 220-240VAC, 50/60Hz
Ambient Temperature Range:	10° C - 40° C (50° F - 104° F)
Power Wattage:	65W @ 11 psi

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

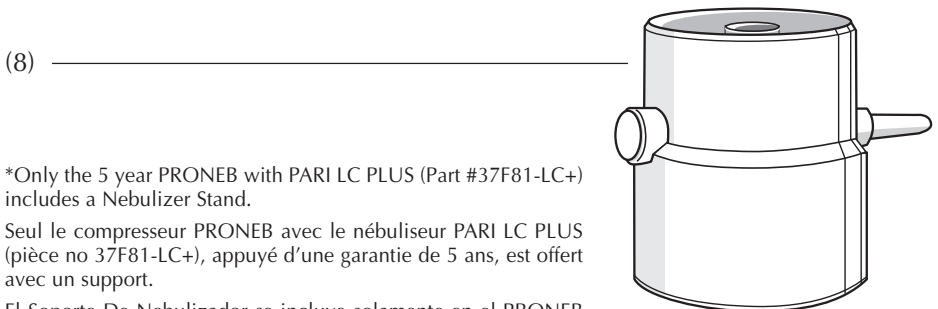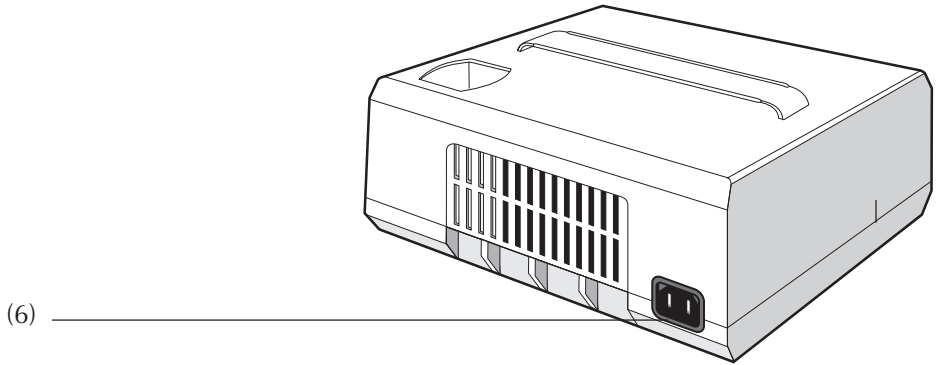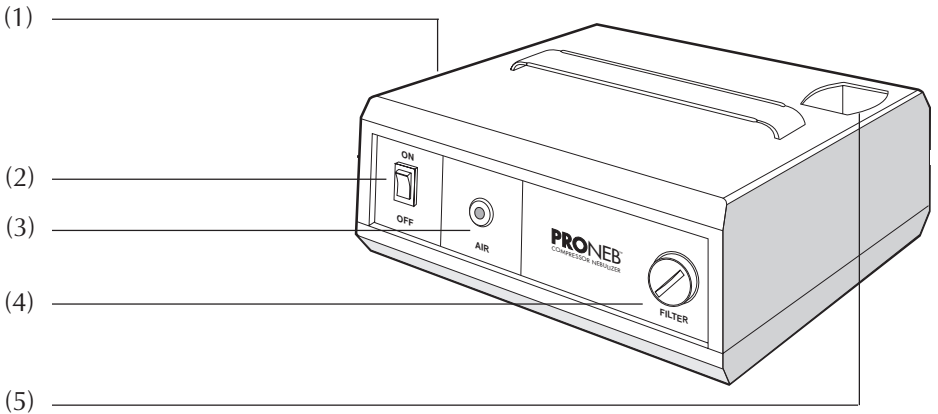
*Only the 5 year PRONEB with PARI LC PLUS (Part #37F81-LC+) includes a Nebulizer Stand.

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I. SAFETY PRECAUTIONS

IMPORTANT SAFEGUARDS

When using electrical and medical products, especially when children are present, basic safety precautions should always be followed, including the following:

READ ALL INSTRUCTIONS BEFORE USING

DANGER

To reduce the risk of serious or fatal injury from electrocution:

1. Do not use or store product where it can fall or be pulled into a bathtub, sink, water, or other liquid. Do not place in or drop into water or other liquid. Do not use while bathing.
2. Do not reach for a product that has fallen into water or other liquid. Unplug immediately.
3. Always unplug this product immediately after using (UL guidelines 1431).

WARNING

To reduce the risk of serious or fatal injury from electrocution, fire or burns and to reduce the risk of damage to the unit and malfunction:

1. Before use, check the specification plate on the bottom of the PRONEB compressor to ensure that the voltage and current indicated on the unit correspond to the voltage and current available.
2. Connect this product to a properly grounded outlet only. See Grounding Instructions on page 4.
3. Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or if it has been exposed to any liquids inside the case. Return the product to an authorized PARI Service Center for examination and repair.
4. Keep all electrical cords away from heated surfaces. Do not overload wall outlets or extension cords.
5. Never block the air openings of this product or place it on a soft surface, such as a bed or couch, where the air openings may be blocked. Keep the air openings free of lint, hair, and similar substances. Blockage could cause excessive heat build-up. A product should never be left unattended when plugged in.
6. Never drop or insert any object into any openings.
7. Be sure the PRONEB is unplugged prior to cleaning.
8. Do not operate outdoors or where Oxygen is being administered in a closed environment such as inside an oxygen tent.
9. Do not open or disassemble the PRONEB Compressor. Refer all servicing to an authorized PARI Service Center.

WARNING

To reduce the risk of serious or fatal injury and improper treatment:

1. Use the product only for its intended use as described in these Instructions For Use. Use only when prescribed by a physician and with the prescribed medication.
2. Do not use nebulizers, tubing or other attachments not recommended by the manufacturer.
3. Close supervision is necessary when this product is used by or near the physically or mentally impaired or children. Never use while sleeping or drowsy.

SAVE THESE INSTRUCTIONS

GROUNDING INSTRUCTIONS

This product should be grounded when used. In the event of an electrical short circuit, grounding reduces the risk of electric shock by providing an escape wire for the electric current. This product is equipped with a cord having a grounding wire with a grounding plug. The plug must be plugged into an outlet that is properly installed and grounded.

⚠ DANGER

Improper use of the grounding plug can result in a risk of electric shock.


If repair or replacement of the cord or plug is necessary, do not connect the grounding wire to either flat blade terminal. The wire with insulation having an outer surface that is green with or without yellow stripes is the grounding wire.

Check with a qualified electrician or serviceman if the grounding instructions are not completely understood, or if in doubt as to whether the product is properly grounded.

This product is for use on a nominal 120 V circuit, and has a grounding plug that looks like the plug illustrated in sketch A. A temporary adapter, which looks like the adapter illustrated in sketches B and C, may be used to connect this plug to a 2-pole receptacle (sketch B) if a properly grounded outlet is not available. The temporary adapter should be used only until a qualified electrician can install a properly grounded outlet (sketch A). The green colored rigid ear, lug, and the like extending from the adapter must be connected to a permanent ground such as a properly grounded outlet box cover. Whenever the adapter is used, it must be held in place by a metal screw.

If it is necessary to use an extension cord, use only a 3-wire extension cord that has a three-blade grounding plug, and a 3-slot receptacle that will accept the plug on the product. Replace or repair a damaged cord.

GROUNDING METHODS

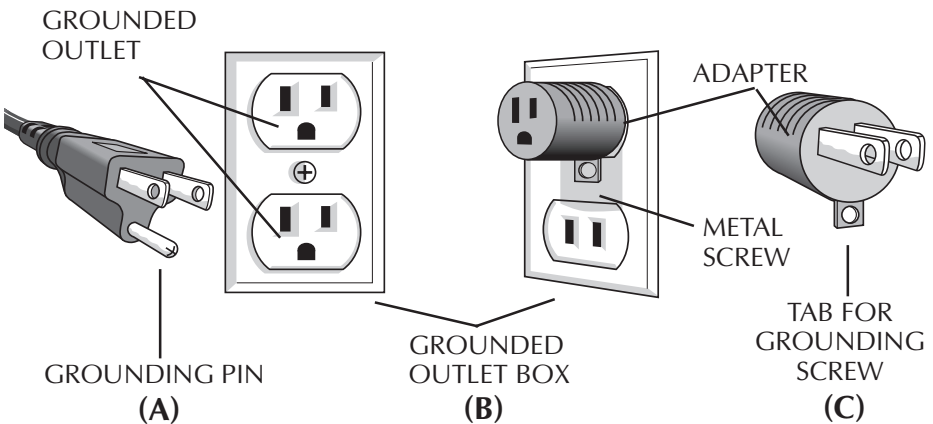II. INSTRUCTIONS FOR USE

These Instructions For Use contain information on the PRONEB Compressor (PRONEB). Separate Instructions For Use are included with the enclosed PARI Nebulizer.

Before using your PRONEB, you should read and understand the entire Instructions For Use for both the PRONEB and PARI Nebulizer. Make special note of all danger, warning, and caution statements. Save this guide for future reference.

III. GETTING STARTED

1. Check the PRONEB packing box and make sure all components are included.

DESCRIPTION:	PART #
PRONEB Compressor	
3 Prong Power Cord for North America	41E8224
Air Filter (in compressor)	41F64
PRONEB Instructions For Use	DS-37-810
Warranty Card (in nebulizer package)	DS-37-9999
Nebulizer Stand (Optional)*	22F1010

PARI Nebulizer Set with Instructions For Use

If you did not receive all of the components, please contact the authorized dealer where you purchased your PRONEB.

2. Fill out the warranty card completely and mail.
3. It is recommended that the patient keep a back-up nebulizer on hand in case parts are lost or damaged.

IV. SET-UP

1. Place the PRONEB on a stable, flat surface. Place the unit in an area that is clean and free of dust (Fig. A).
2. Connect power cord to socket (Fig. B) on the back of the PRONEB and connect power cord plug to power outlet (wall outlet).
3. Open the nebulizer package and read and follow the nebulizer's Instructions For Use.

*Note: The 5 year Warranty PRONEB Compressor (see unit Serial Number to verify warranty coverage) may contain a nebulizer stand for the enclosed nebulizer. The nebulizer stand attaches to the bottom of the nebulizer (Fig. C). The nebulizer stand allows the nebulizer to be placed in the nebulizer holder on top of the PRONEB with the tubing attached.

V. MAINTENANCE OF THE PRONEB

WARNING

To reduce the risk of increased bacterial growth, infection, illness, or injury from contamination: at the end of each treatment, thoroughly clean and dry all parts of the nebulizer and thoroughly dry any moisture or condensation in the tubing, following the instructions below.

V. MAINTENANCE OF THE PRONEB

Tubing Maintenance

If there is any moisture or condensation in the tubing after the treatment, let the compressor run with only the tubing attached for 10 minutes. The warm air from the compressor will dry the moisture or condensation in the tubing. Alternatively, you can dry the tubing by removing it from the compressor and hanging it up with the “ends down” (Fig. D) to allow any moisture or condensation to drain out and evaporate. Use a damp cloth to wipe the exterior of the tubing.

Cleaning the PRONEB Housing

Use a damp cloth to wipe the exterior of the PRONEB housing and tubing (Fig. E). Never spray fluid into the ventilation slots on the PRONEB Compressor. Fluid could cause damage to the electrical and compressor parts and could lead to a malfunction. In the event that fluid enters the unit, contact a PARI authorized service center.

Filter Replacement

Under normal conditions, the filter on the front panel of the PRONEB should be replaced after 500 hours of use, or sooner if it appears soiled (gray or brown), clogged or wet. Check the condition of the filter after every 10-12 treatments. Do not attempt to wash and re-use filters. Do not use substitute filter material. Compressor damage can result. Turn the round plastic filter cover (Fig. F) counterclockwise (left) and remove filter (use a coin to help turn the cover if tight) (Fig. G). Replace with part #41F64.

Maintenance and Repair

Never operate the PRONEB if it is damaged or improperly or incompletely assembled. Use only original PARI parts and attachments authorized by PARI for the PRONEB. Use of other parts and attachments may damage the unit or cause improper treatment. Do not open or disassemble the compressor unit at any time. Repairs, service and maintenance should be carried out only by authorized PARI service centers.

Note: There are no user serviceable parts inside the PRONEB compressor case. Service by anyone other than authorized PARI Service Center voids warranty.

VI. WARRANTY OR REPAIR RETURN PROCEDURE

1. Contact PARI Patient Connection (Service Department) using the phone numbers listed in Section VII (Proneb Limited Warranty) of this booklet.
2. In writing, describe the malfunction or symptoms of the PRONEB. Include your full name, address and daytime telephone number.
3. Send written description and PRONEB in a protected box to the address provided by Patient Connection.

VII. PRONEB LIMITED WARRANTY

PARI offers a limited warranty on the PRONEB Compressor and its parts. The duration of this warranty ("Warranty Coverage Period") depends on the Proneb Compressor model as follows:

Model #44B1100 = 3 Years

Model #44V0000 = 5 Years

Model #37-0101 = 5 Years

If you are not sure of the model number of your PRONEB, refer to the Serial Number Label on the bottom of the unit.

PARI warrants the PRONEB compressor and its parts will be free from defects in workmanship and materials for the warranty coverage period, starting from date of first retail purchase of the equipment. This warranty does not cover failures of the nebulizer, tubing, or accessories. This warranty does not extend to failures resulting from accident, misuse, abuse, alteration, use of unauthorized service, parts or cleaning solutions, or failure to comply with Instructions for Use.

If the warranted equipment should fail during the warranty coverage period, PARI, at its option, will repair or replace the equipment or parts at issue, provided PARI determines to its satisfaction the claim is bona fide. It shall be the responsibility of the purchaser to pack and return the equipment in a manner to avoid shipping damage. The shipment must be prepaid to PARI Respiratory Equipment, Inc. This warranty covers the cost of labor incurred in the removal or replacement of the warranted component parts at authorized PARI service centers only. To make claim under this warranty, notify the authorized service center by calling PARI at:

1.800.FAST.NEB (327.8632), Tel. 804.639.7235, Fax 804.639.7242

THERE ARE NO OTHER EXPRESS WARRANTIES. ALL IMPLIED WARRANTIES ON THE PRONEB AND ITS PARTS, INCLUDING IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED TO WARRANTY COVERAGE PERIOD, STARTING FROM THE DATE OF FIRST RETAIL PURCHASE. ALL EXPRESS OR IMPLIED WARRANTIES ON ANY OTHER PARTS, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE DISCLAIMED. Some states do not allow time limitations on implied warranties, so the above limitations may not apply to your situation. The sole and exclusive remedy under this warranty is to repair or replace the PRONEB and/or its parts at the option of PARI. IN NO EVENT SHALL PARI BE LIABLE FOR INCIDENTAL OR CONSEQUENTIAL DAMAGES TO THE PURCHASER OR ANY THIRD PARTY, INCLUDING WITHOUT LIMITATION, DAMAGES ARISING FROM PERSONAL INJURIES, LOST PROFITS, LOSS OF BUSINESS OPPORTUNITY, LOSS OF PROPERTY, ECONOMIC LOSSES, OR STATUTORY OR EXEMPLARY DAMAGES, WHETHER IN NEGLIGENCE, WARRANTY, STRICT LIABILITY OR OTHERWISE. Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitations or exclusions may not apply to you. This warranty gives you specific rights. You may also have other legal rights, which vary, from state to state.

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