


PARI TREK™


COMPRESSOR NEBULIZER SYSTEM

INSTRUCTIONS FOR USE

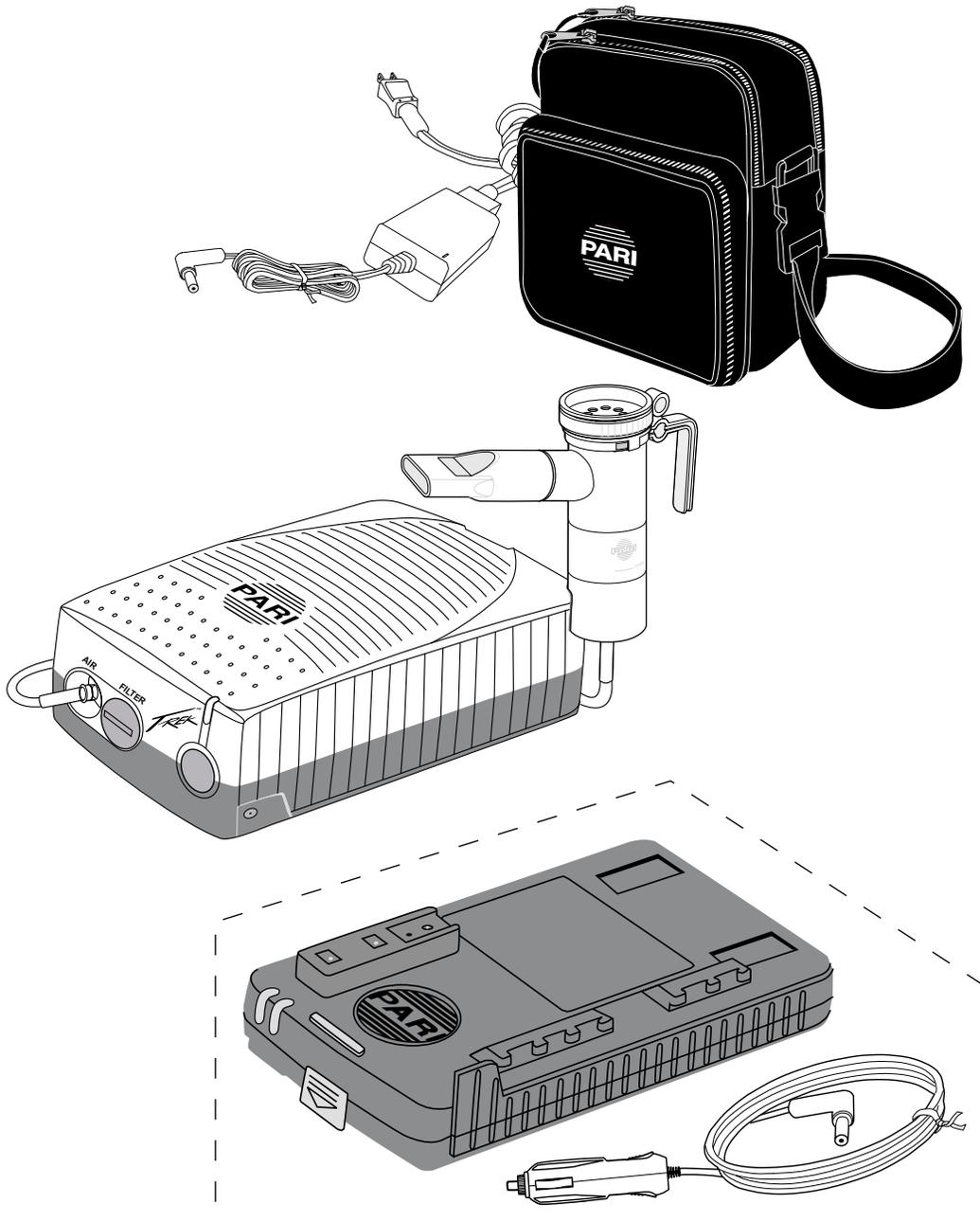


Portable Power Kit (optional)

Multi-Voltage AC Adapter

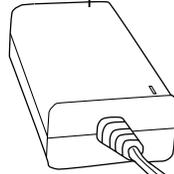


Power Cord

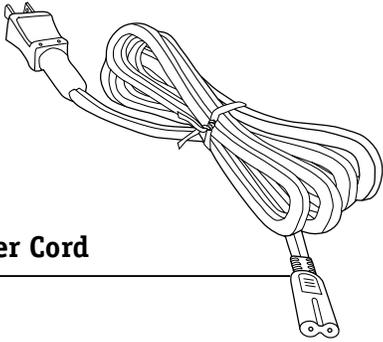

Carrying Case


Tubing

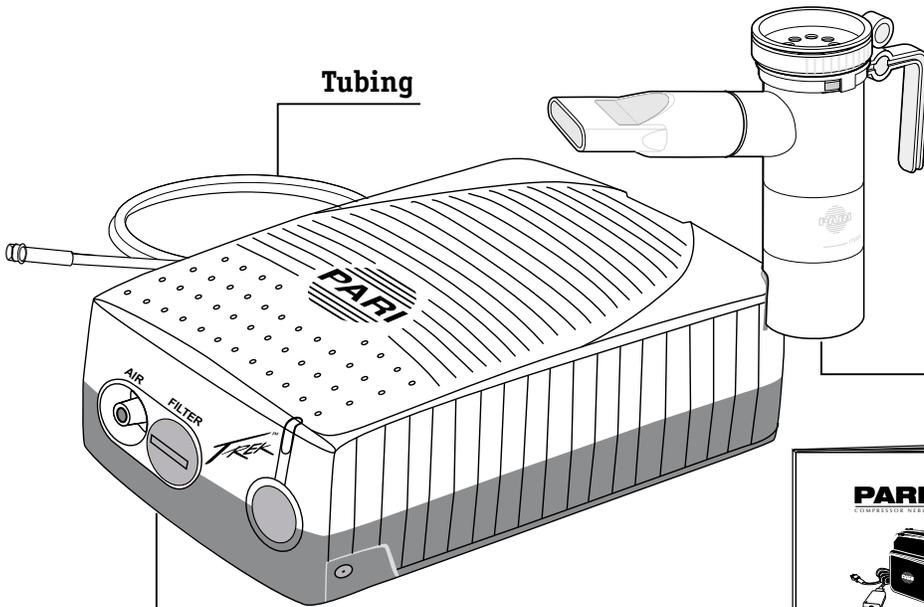


PARI Reusable Nebulizer

PARI TREK Compressor

Instructions For Use

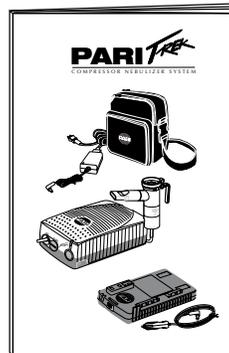





**Portable Power Kit
(optional accessory)
Part# 46F25**

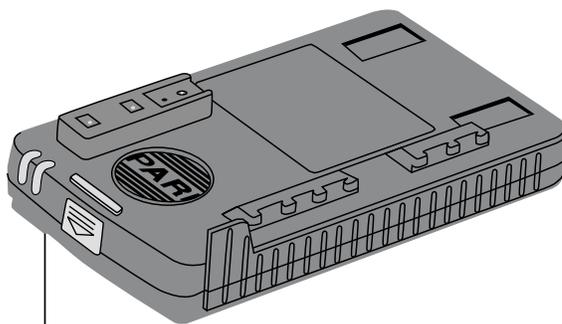

PARI TREK Rechargeable Battery (optional accessory)

**12V DC Adapter
(optional accessory)**

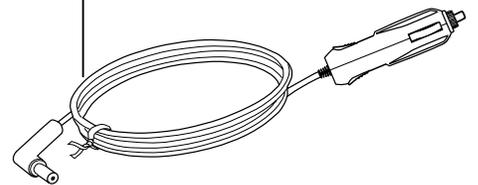

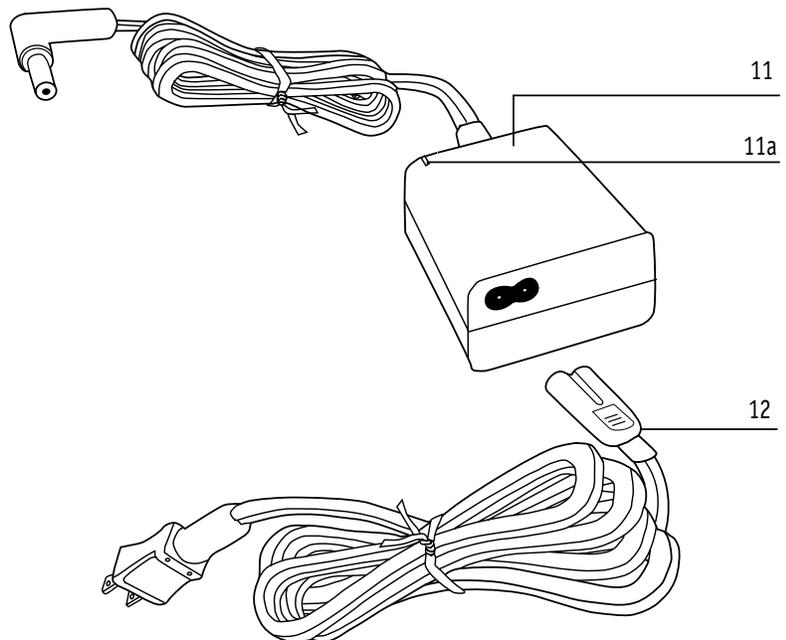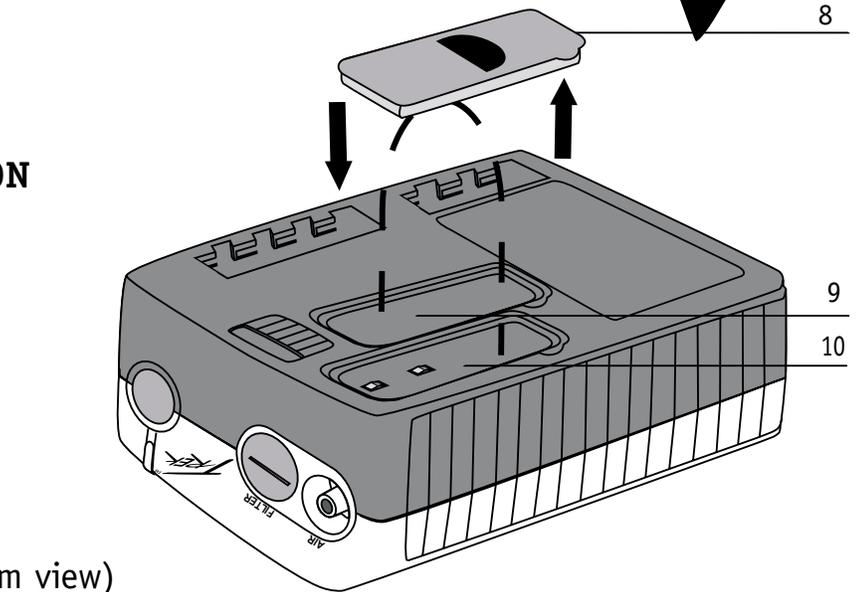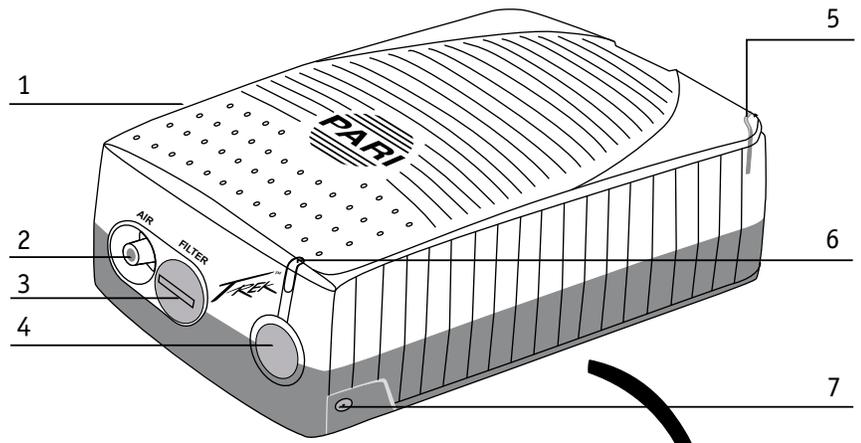


1. PARI TREK COMPRESSOR ILLUSTRATION

- 1 PARI TREK Compressor
- 2 Air Outlet
- 3 Air Filter
- 4 On/Off Switch
- 5 Nebulizer Holder
- 6 Multi-Color Status Indicator
- 7 Power Receptacle
- 8 Removable Connection Cover (bottom view)
- 9 Removable Connection Cover Storage (bottom view)
- 10 Battery Connection (bottom view)
- 11 Multi-Voltage AC Adapter
- 11a Power Indicator
- 12 Power Cord

2. PARI REUSABLE NEBULIZER ILLUSTRATION

- 1 Inspiratory Valve Cap
- 2 Nebulizer Insert
- 3 Nebulizer Semi-Circle
- 4 Mouthpiece with Expiratory Valve
- 5 Nebulizer Outlet
- 6 Nebulizer Cup
- 7 Air Intake
- 8 Tubing

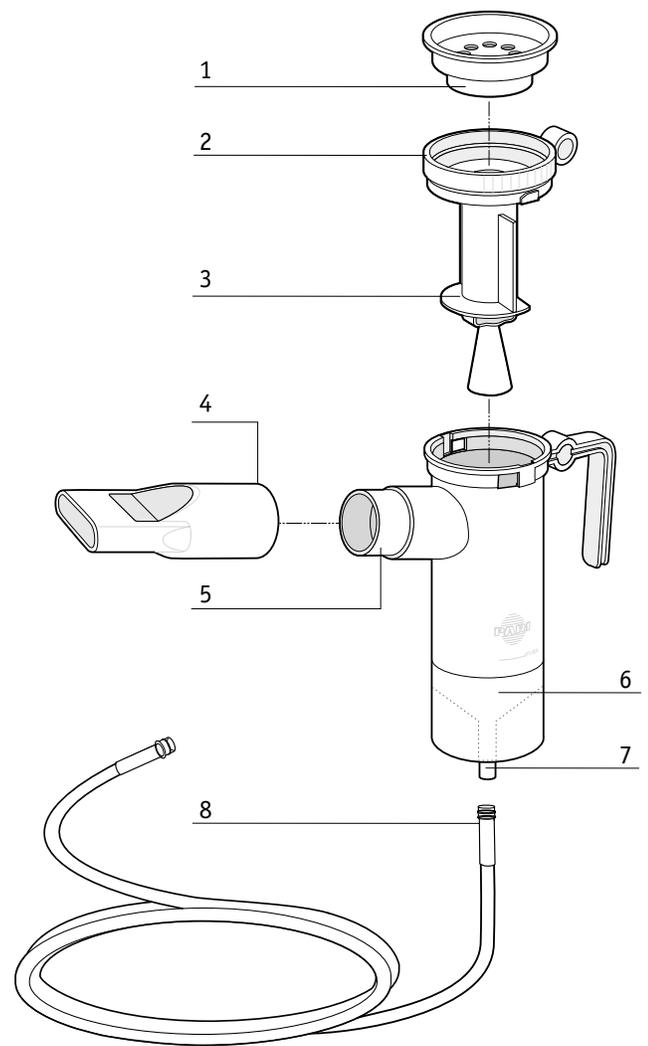

3. PARI TREK RECHARGEABLE BATTERY AND 12V DC ADAPTER ILLUSTRATION (OPTIONAL PORTABLE POWER KIT)

- 1 Charge Indicator
- 2 Battery Level Indicator
- 3 Release Button
- 4 Charge Receptacle
- 5 Battery Contacts
- 6 12V DC Adapter

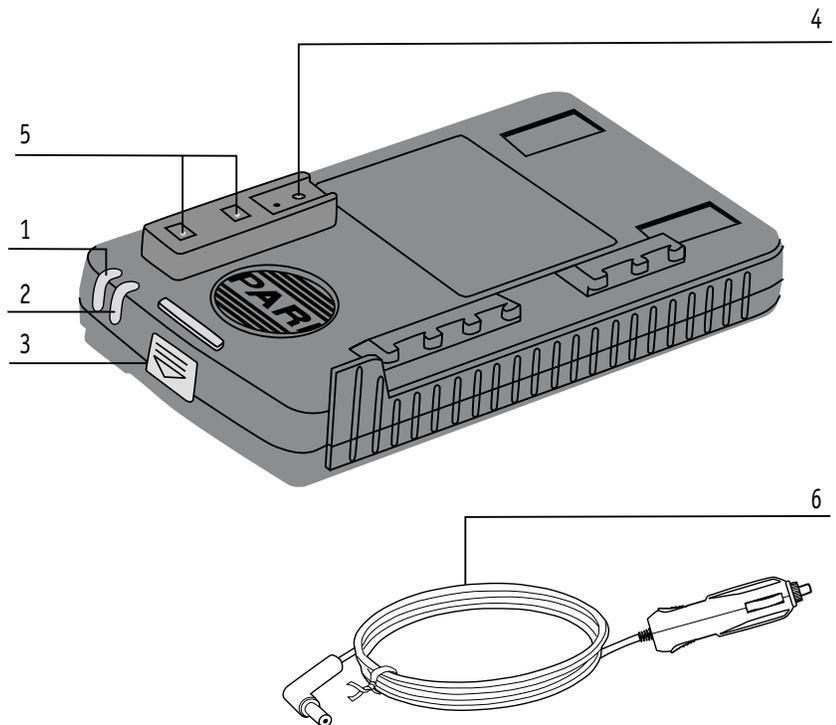


FIGURE A

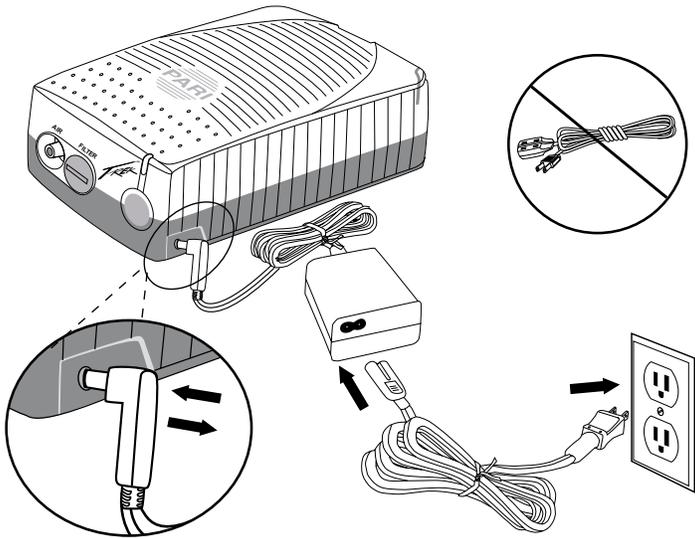


FIGURE B

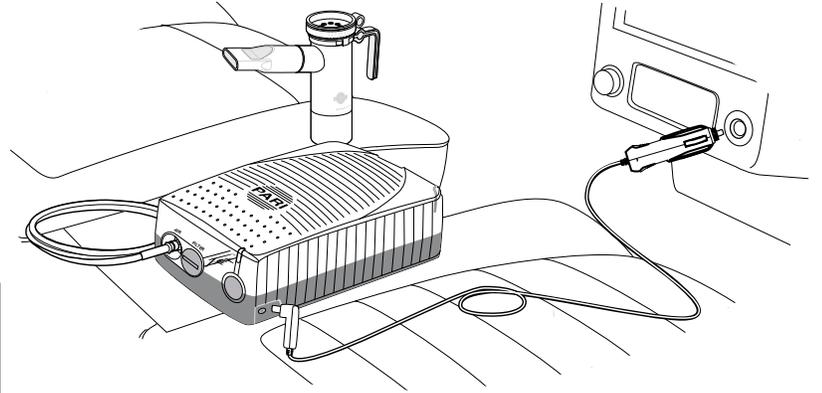


FIGURE C

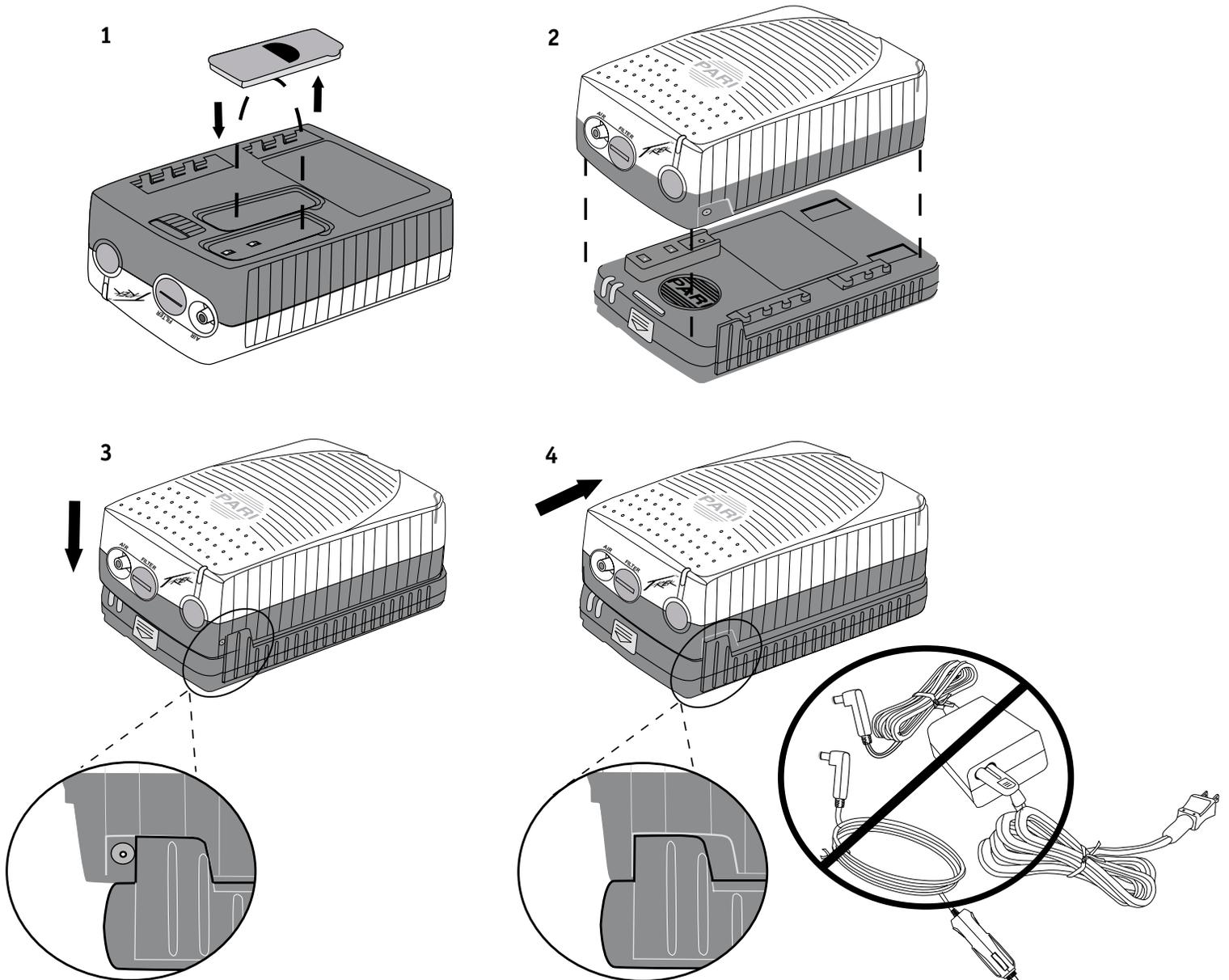


FIGURE D

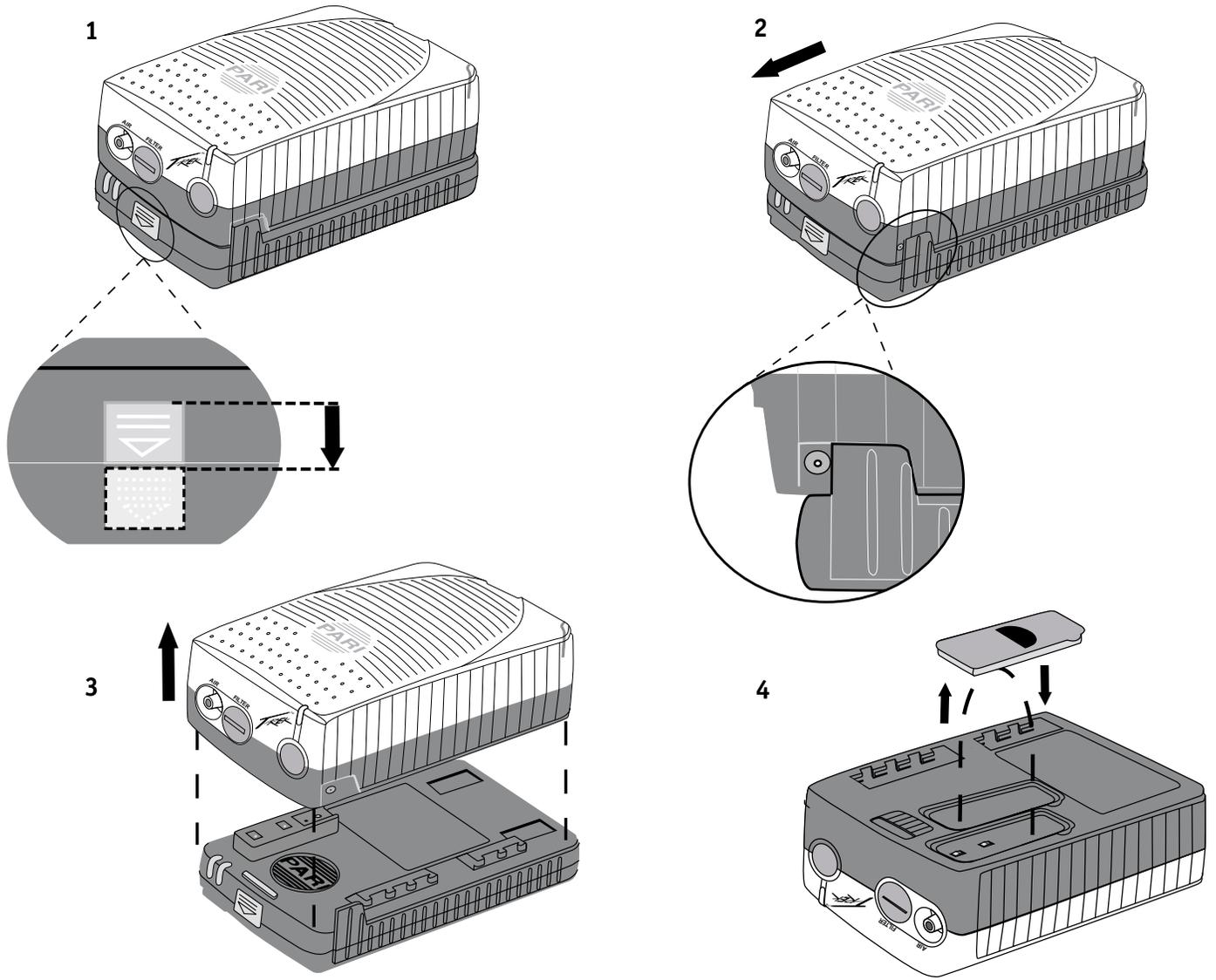


FIGURE E

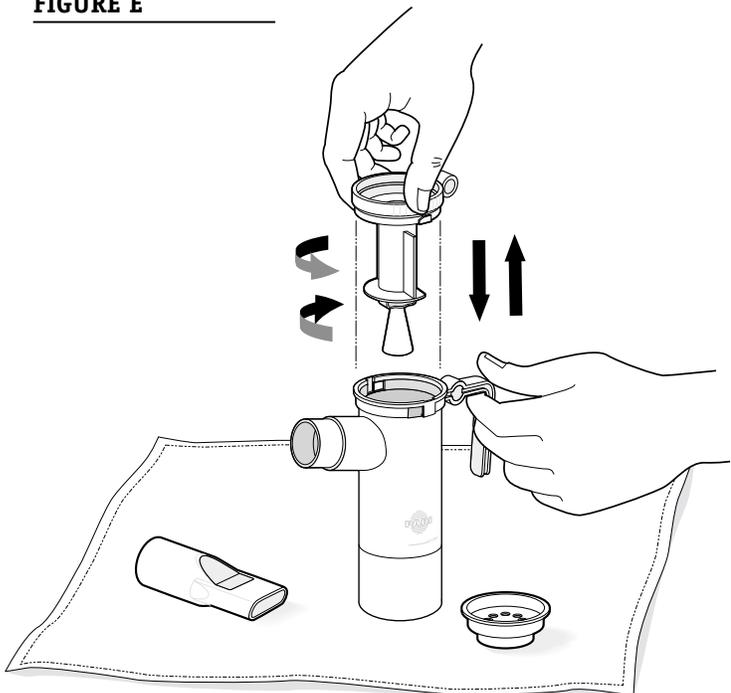


FIGURE F

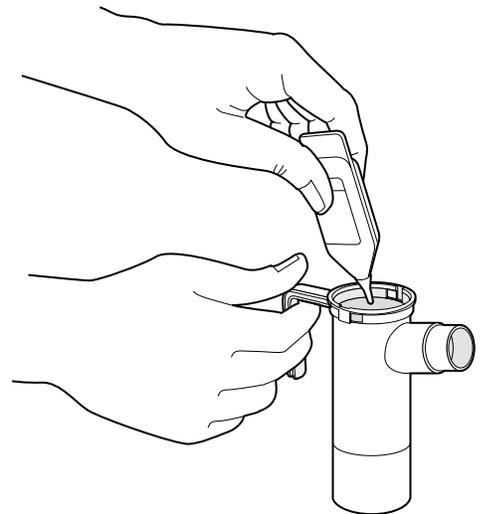


FIGURE G

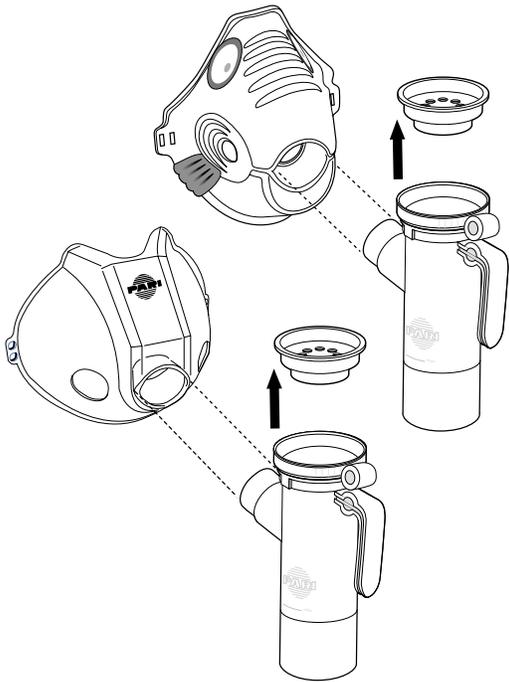


FIGURE H

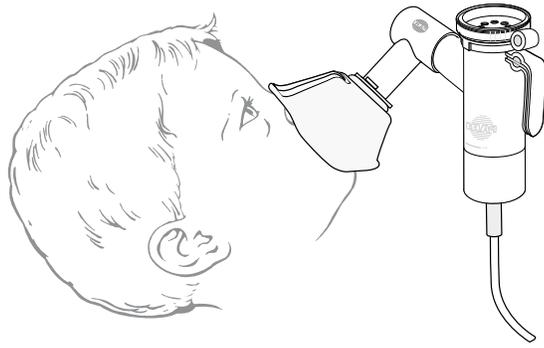


FIGURE J

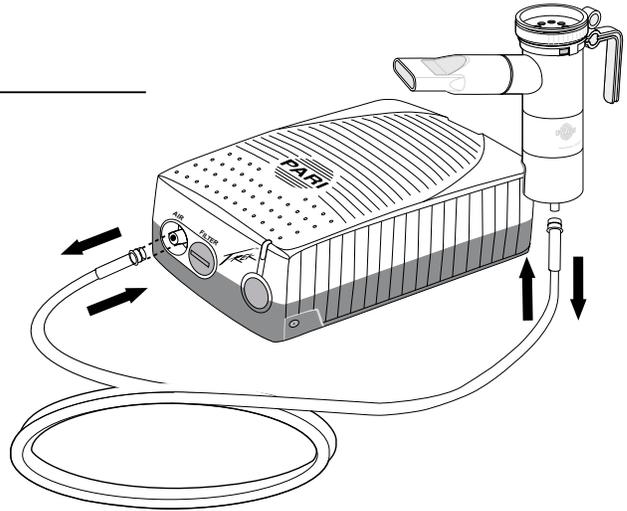


FIGURE K

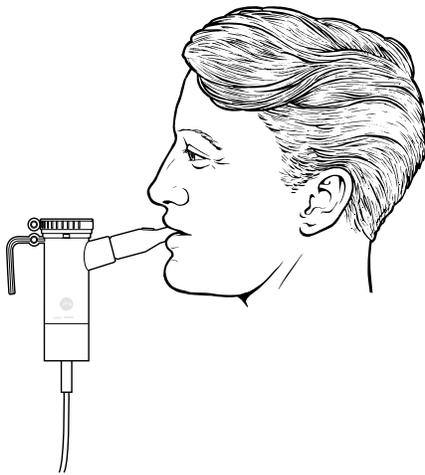


FIGURE L







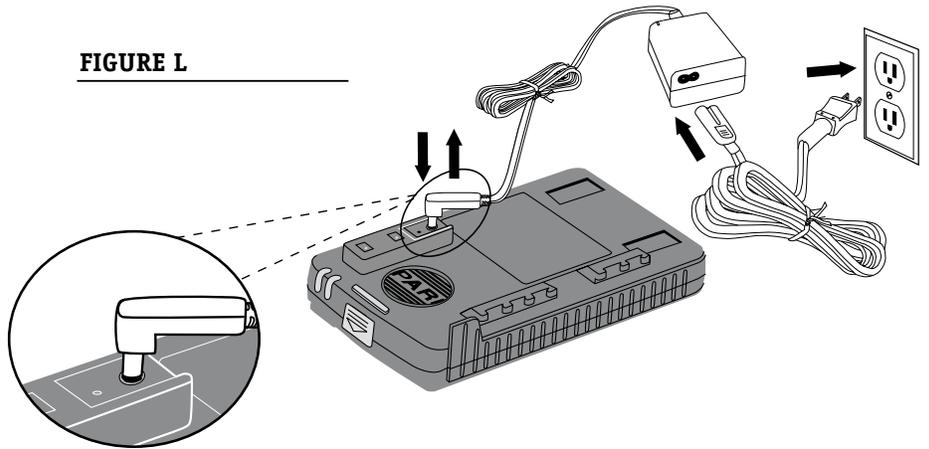


FIGURE M

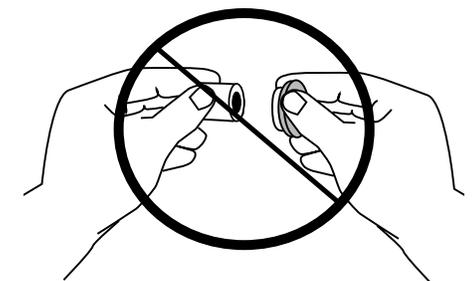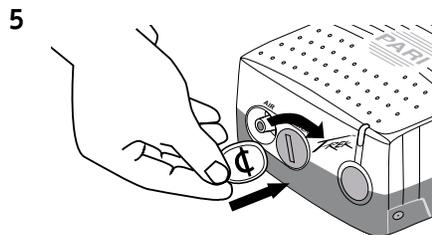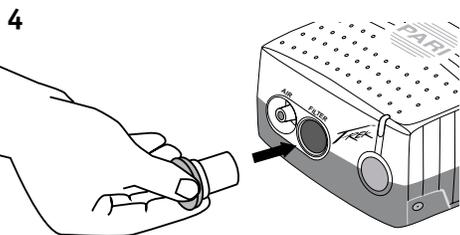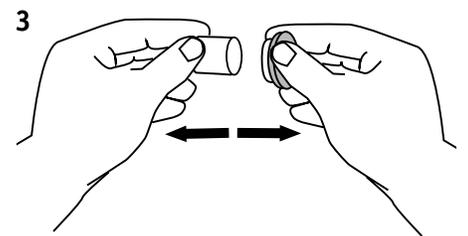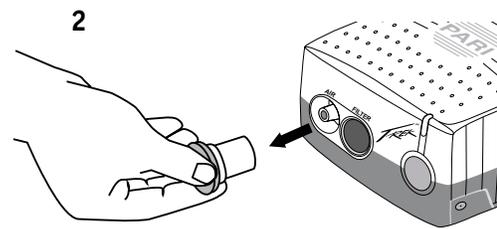

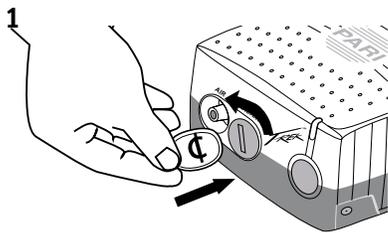

Instructions For Use

		Page
1	PARI TREK Compressor Illustration2
2	PARI Reusable Nebulizer Illustration3
3	PARI TREK Battery & 12V DC Adapter Illustration3
	Other Illustrations4-6
4	Introduction8
5	Safety Precautions	
	Compressor Safety Precautions9
	Nebulizer Safety Precautions11
	Battery Safety Precautions12
6	Getting Started14
7	Compressor	
	Set Up15
	Operation From a Wall Power Source15
	Operation Using 12V DC Adapter16
	Operation Using the Rechargeable Battery16
8	Reusable Nebulizer	
	Set Up18
	Treatment19
	Cleaning21
	Disinfecting and Sterilizing22
9	Rechargeable Battery	
	Charging23
	Storing24
10	PARI TREK Compressor	
	Maintenance24
	Specifications26
	Limited Warranty27
	Service and Repair28

PARI TREK COMPRESSOR INSTRUCTIONS FOR USE

4. INTRODUCTION

A LARGER TEXT VERSION OF THESE INSTRUCTIONS FOR USE IS AVAILABLE FOR VISUALLY IMPAIRED PATIENTS FREE OF CHARGE AT WWW.PARI.COM/PRODUCTSUPPORT/INSTRUCTIONS.HTML

These instructions contain important information and safety precautions on the PARI TREK Compressor Nebulizer System.

- Illustrations of the proper set up of the PARI TREK Compressor are on page 2.
- Illustrations of the proper set up of the PARI Reusable Nebulizer are on page 3.
- Illustrations of the Rechargeable Battery and 12V DC Adapter for the PARI TREK Compressor are on page 3.
- Open the fold-out pages in the front of these **Instructions For Use** to view the illustrations while reading the instructions.

Before using your PARI TREK Compressor Nebulizer System, you should read and understand the entire **Instructions For Use** for the PARI TREK Compressor, PARI Reusable Nebulizer and optional portable power kit. Take special note of all safety precautions marked DANGER and WARNING. **Save these instructions for future reference.**

5. SAFETY PRECAUTIONS

IMPORTANT SAFEGUARDS

READ ALL WARNINGS BEFORE USING YOUR PARI TREK COMPRESSOR NEBULIZER SYSTEM, BATTERY AND 12V DC ADAPTER.



Caution: U.S. Federal Law restricts this device to sale by or on the order of a physician. Use only when prescribed by a physician and with the prescribed medication.

Read all warnings and instructions before use. Use the product only for its intended use as described in these **Instructions for Use**. Improper use can result in serious or fatal illness/injury, improper treatment or property damage.

Use only with nebulizers, tubing and accessories authorized by PARI for this device. Other nebulizers, tubing and accessories may result in improper treatment. Other compressors and tubing may produce incorrect liter flow, resulting in improper treatment. Use with other compressors, tubing, nebulizers and accessories should be reviewed with your healthcare professional.

COMPRESSOR SAFETY PRECAUTIONS



To reduce the risk of serious or fatal injury from electrocution:

1. Do not place or store the product where it can fall or be knocked into bathtub, sink, water, or other liquid. Do not place or drop into water or other liquid. Do not use while bathing.

2. Do not reach for a product that has fallen into water or other liquid. Unplug immediately.
3. Always unplug immediately after using.



To reduce the risk of serious or fatal injury from electrocution, fire, burns and to reduce the risk of damage and malfunction of the unit:

1. Before use, check the serial number label on the bottom of the compressor, battery and on the Multi-Voltage AC Adapter to ensure that the voltage and current indicated on the unit correspond with the voltage and current available.
2. Never operate this product if it:
 - has a damaged cord or plug,
 - is not working properly,
 - has been dropped or damaged,
 - has been exposed to any liquids inside the case.

Return non-operating product to an authorized PARI Service Center for examination, repair or replacement.

3. Keep all electrical cords away from heated surfaces. Do not overload wall outlets or use extension cords.
4. Do not operate where oxygen is being administered in a closed environment such as an oxygen tent.
5. Never drop or insert any object into any openings.
6. Be sure the PARI TREK Compressor and the Multi-Voltage AC Adapter are unplugged prior to cleaning.

7. Do not open or disassemble the PARI TREK Compressor. Refer all servicing to an authorized PARI Service Center.
8. Use only adapters that are authorized for the PARI TREK Compressor. Use of unapproved accessories can lead to improper treatment or damage to the compressor, adapters, or the battery.
9. Never spray liquids onto the compressor housing or battery. Fluid could cause damage to the electrical and compressor parts and could lead to a malfunction. In the event that fluids enter the unit, return the product to an authorized PARI Service Center for examination and repair.
10. Do not use the PARI TREK Compressor or the PARI Reusable Nebulizer while operating a vehicle.

NEBULIZER SAFETY PRECAUTIONS



1. Nebulizer is for single patient use (one person) only. Do not share your nebulizer with other patients.
2. To reduce the risk of increased bacterial growth, infection, illness, or injury from contamination, thoroughly clean and dry all parts of the nebulizer and thoroughly dry any moisture or condensation in the tubing at the end of every treatment, following the instructions below.
3. Cleaning the nebulizer after each treatment is not enough---you must also regularly disinfect or sterilize the nebulizer between treatments to prevent serious or fatal illness caused by contamination of the nebulizer.
4. Never use a nebulizer with a clogged nozzle. If the nozzle is clogged, the aerosol mist will be reduced, altered or stopped, which will diminish or eliminate the effectiveness of the treatment. If clogging occurs, replace with the same model PARI Reusable Nebulizer.

5. Close supervision is necessary when this product is used by or near children and physically or mentally impaired persons. Never use when sleeping or drowsy.
6. This product contains small parts that may present a choking hazard to small children. The tubing also presents a strangulation hazard. **ALWAYS USE CLOSE ADULT SUPERVISION WHEN ADMINISTERING A TREATMENT TO A CHILD. DO NOT LEAVE CHILD UNATTENDED DURING TREATMENT.**
7. Children under the age of 3 or any patient who is unable to use a mouthpiece properly under supervision should use a face mask.
8. Before use, check PARI Reusable Nebulizer for proper assembly. All parts should be seated firmly in place. Use of an improperly assembled nebulizer could diminish or prevent adequate delivery of medication and the effectiveness of the treatment.

BATTERY SAFETY PRECAUTIONS



1. Use only the PARI TREK Rechargeable Battery described in these **Instructions For Use** for operating your PARI TREK Compressor.
2. Do not immerse the PARI TREK Rechargeable Battery in water.
3. Do not touch the battery with wet or moist hands.
4. Do not expose battery to direct sunlight, humidity, extreme temperatures or chemicals.
5. Make sure battery poles do not come into contact with keys or other metal objects, causing a "short circuit".

6. Battery is protected from overheating. Battery may not operate if temperature is 104°F (40°C) or higher. Allow battery to cool before restarting.
7. To charge the PARI TREK Rechargeable Battery use only the PARI TREK Multi-Voltage AC Adapter (part number 46F0002) or optional 12V DC Adapter (part number 46F0010). Using other adapters may damage the battery.
8. In case of failure in charging or operating, unplug the Rechargeable Battery from the Multi-Voltage AC Adapter and turn the unit off. Should the Rechargeable Battery fail to charge or operate improperly, even if the Instructions For Use are followed, notify the authorized service center by calling PARI Patient Connection toll free (USA) at 1.877.943.3931, then press 1.
9. Do not attempt to open the plastic case or to replace any components. Do not disassemble Rechargeable Battery. The Rechargeable Battery is a sealed unit and can not be repaired.
10. Protect the environment: when replacing the Rechargeable Battery, recycle old battery or dispose of properly in accordance with local laws.
11. Do not incinerate (burn).

SAVE THESE INSTRUCTIONS

6. GETTING STARTED

1. Check the box to make sure the following basic components of the PARI TREK Compressor Nebulizer System are enclosed:

- PARI TREK Compressor
- Multi-Voltage AC Adapter
- Power Cord
- PARI LC® Reusable Nebulizer (**See page 3** for illustration of parts)
- Tubing
- Carrying Case
- PARI TREK Compressor Nebulizer System **Instructions For Use** with warranty card
- Patient Instructional Video

If you did not receive the components listed, please contact the dealer where you purchased your PARI TREK Compressor Nebulizer System.

2. Completely fill out the postage-paid warranty card on the back cover and mail.

3. It is always a good idea to keep an extra PARI Reusable Nebulizer (part number 22F81 for PARI LC Plus® or 22F51 for PARI LC Star®) and an extra PARI TREK Compressor Air Filter (part number 41E4851) on hand at all times. You may need them if parts become lost, damaged, or soiled.

7. COMPRESSOR

COMPRESSOR SET UP

1. Place the PARI TREK Compressor on a stable flat surface. Place the unit in an area that is clean and free of dust.

NOTE: Check the filter monthly or every 10-12 treatments and replace immediately if dirty, clogged, or wet to ensure proper operation of the compressor.

OPERATION FROM A WALL POWER SOURCE

1. Plug the Power Cord into the wall outlet. Connect the other end of the Power Cord into the Multi-Voltage AC Adapter. Power Indicator on Multi-Voltage AC Adapter will illuminate green. Next, plug the Multi-Voltage AC Adapter into the Power Receptacle on the side of the PARI TREK Compressor. Refer to page 4 figure A.
2. The PARI TREK Multi-Voltage AC Adapter (part number 46F0002) automatically adjusts to the incoming voltage (100-240V) so that it can be used in most countries. For use internationally, plug pattern adapters (part number 44F2207 Adapter Plug Set) may be needed to plug your Power Cord into the wall outlet. See voltages listed on the Multi-Voltage AC Adapter label for more specific information.
3. The Multi-Color Status Indicator on the PARI TREK Compressor will illuminate green when the compressor is operating.

OPERATION USING 12V DC ADAPTER *(optional accessory)*:

NOTE: Do not use the PARI TREK Compressor or the PARI Reusable Nebulizer while operating a vehicle.

1. Plug the 12V DC Adapter into the vehicle outlet and plug the other end of the adapter into the Power Receptacle on the side of the PARI TREK Compressor. Refer to page 4 figure B.

NOTE: Best results will be achieved when the car engine is running.

2. The Multi-Color Status Indicator on the PARI TREK Compressor will illuminate green when the compressor is operating.

OPERATION USING THE RECHARGEABLE BATTERY *(optional accessory)*:

1. Completely charge the battery before using for the first time. Refer to **Charging the Battery** on page 23.
2. The Rechargeable Battery can not be connected to the PARI TREK Compressor if either the Multi-Voltage AC Adapter or 12V DC Adapter are plugged into Power Receptacle. Refer to page 4 figure C.
3. To uncover the Battery Connection, remove the blue Removable Connection Cover from the bottom of the compressor. Refer to page 4 figure C.
4. Set the blue Removable Connection Cover into the storage area provided in the bottom of the PARI TREK Compressor. Refer to page 4 figure C.
5. Attach the PARI TREK Compressor to the Rechargeable Battery by placing it on top of the battery and sliding it into place. Refer to page 4 figure C. There will be an audible click once the battery is locked into place. Ensure battery is securely locked onto the compressor before use.

6. The Multi-Color Status Indicator on the PARI TREK Compressor will illuminate green when the compressor is operating.
7. Use the Battery Charge Indicator light to determine the condition of the Battery:
 - GREEN – Battery is plugged into the wall outlet and is fully charged.
 - ORANGE – When blinking, Battery must be recharged. Use other power source using Multi-Voltage AC Adapter or 12V DC Adapter.

When the battery is completely discharged all indicators on the battery and compressor will extinguish and the compressor will shut off. Refer to **Charging the Battery** on page 23.

8. To remove the Battery from the PARI TREK Compressor, push and hold the Release Button down on the front of the Battery then slide the Battery forward and lift the compressor off the Battery. Refer to page 5 figure D.
9. To protect the Battery Connection from possible damage, attach the blue Removable Connection Cover to the bottom of the PARI TREK Compressor. Refer to page 5 figure D.
10. The Rechargeable Battery can not be recharged while it is attached to the PARI TREK Compressor. Refer to page 6 figure L.

8. NEBULIZER

NEBULIZER SET UP (Note: Nebulizer is not sterile)

1. Make sure the nebulizer has been cleaned and/or disinfected before first use and after last treatment of the day. Refer to the **Cleaning and Disinfecting the Nebulizer** section on pages 21 and 22.
2. Remove the Nebulizer Insert by twisting counter-clockwise until it stops and lift up. Refer to page 5 figure E.
3. Pour prescribed medication into Nebulizer Cup. Refer to page 5 figure F. Do not fill more than 8ml per treatment for PARI LC Plus® Reusable Nebulizer (or 6ml for PARI LC Star® Reusable Nebulizer).
4. Replace the Nebulizer Insert and make sure the semi-circle is facing the nebulizer outlet and the two circles should be next to one another. Refer to page 5 figure E.
5. If using an optional mask, attach the mask directly to the Nebulizer Outlet. Refer to page 6 figure G and H.

NOTE: When using face mask, the Inspiratory Valve Cap must be either in place or removed depending on the type of mask. See chart for proper position. Refer to page 6 figure G & H.

Mask Type	Part #	Remove Inspiratory Valve Cap	Leave Inspiratory Valve Cap in place
Pediatric Rigid Aerosol Mask	14F15-PP	X	
Adult Aerosol Mask	14F20	X	
Bubbles the Fish™ Pediatric Mask	44F7240	X	
PARI Baby™ Size #0 Nebulizer Set	22F90		X
PARI Baby™ Size #1 Nebulizer Set	22F91		X
PARI Baby™ Size #2 Nebulizer Set	22F92		X
PARI Baby™ Size #1 Mask Conversion Kit	44F2401		X
PARI Baby™ Size #2 Mask Conversion Kit	44F2402		X
PARI Baby™ Size #3 Mask Conversion Kit	44F3301		X

6. If using the Mouthpiece, attach the Mouthpiece to the Nebulizer Outlet with the Expiratory Valve facing up. Then attach the Inspiratory Valve Cap by pressing firmly into the Nebulizer Insert. **The Inspiratory Valve Cap is designed to fit tightly.** Refer to PARI Reusable Nebulizer Illustration on page 3 item 1.
7. Press one end of the tubing firmly onto the Air Outlet on the front of the PARI TREK Compressor. Refer to page 6 figure J. It is important that the tubing be pushed firmly all the way onto the Air Outlet of the PARI TREK Compressor. The tubing is designed to fit tightly.
8. Firmly press the other end of tubing onto the Air Intake on the bottom of the PARI Reusable Nebulizer. Refer to page 6 figure J.

NEBULIZER TREATMENT

1. Take your treatment as directed by your physician.
2. The PARI Reusable Nebulizer has 2 valves that open and close during the nebulizer treatment when the Mouthpiece is used. These valves help to maximize aerosol delivery and minimize medication waste.
3. Sit in a relaxed, upright position and press the blue On/Off Switch (refer to page 2) once to turn on the PARI TREK Compressor. The Multi-Color Status Indicator on the front of the compressor will illuminate green.
4. Place the Mouthpiece on top of your tongue and close your lips around the Mouthpiece. Take slow deep breaths to inhale the aerosol medication slowly. Refer to page 6 figure K. As you inhale, the valve on the Inspiratory Valve Cap opens.
5. Exhale slowly through your mouth. As you exhale, the Expiratory Valve on the Mouthpiece opens, releasing your exhaled aerosol. At the same time, the Inspiratory Valve closes on the top of the nebulizer which reduces waste.

6. If using a mask, place the mask over the nose and mouth and inhale and exhale slowly.
7. Continue steps 4 and 5 until the medication is gone or you hear a "sputtering". Press the blue On/Off Switch once to turn off the PARI TREK Compressor.

If the Multi-Color Status Indicator (refer to page 2 item 6) on the front of the PARI TREK Compressor turns red and the compressor shuts off, then the PARI TREK Compressor has reached its temperature limit and automatically turned off to prevent damage to the PARI TREK Compressor. In the unlikely event this occurs, the PARI TREK Compressor will need approximately 30 minutes to cool down prior to resuming the treatment. Press the blue On/Off Switch once to turn off the PARI TREK Compressor and wait approximately 30 minutes for the PARI TREK Compressor to cool down before restarting the compressor. If the PARI TREK Compressor does not start please allow another few minutes for the compressor to cool and try again.

NOTE: The PARI TREK Compressor will not automatically come back on once the unit has cooled down.

TREATMENT TIMES:

- Patients who use a 2.5ml volume fill will typically find that the PARI TREK Compressor will deliver the treatment in less than 10 minutes. Treatment time depends upon breathing volume, patient's age and disease state.
- The PARI TREK Compressor was designed for intermittent use to deliver aerosol medications, and not for continuous use humidification or prolonged treatments (not over 30 minutes at one time). The PARI TREK Compressor is equipped with a re-setable thermal protector to prevent damage to the PARI TREK Compressor unit. If the thermal protector cuts off, the PARI TREK Compressor will shut down until it is cool (approximately 30 minutes). Then treatments can continue.

- If you will be using the PARI TREK Compressor for treatments longer than 30 continuous minutes at one time, please contact PARI Patient Connection at 1.877.943.3931 (USA) then press 1 for tips on prolonged use.

CLEANING THE PARI REUSABLE NEBULIZER

1. Disconnect Tubing from Air Intake on the bottom of the Nebulizer Cup. Refer to page 6 figure J.
2. If there is any moisture (condensation) in the Tubing, let the PARI TREK Compressor run with only the Tubing attached for 2-3 minutes. The warm air from the PARI TREK Compressor will dry out the Tubing. Alternatively, you can dry the Tubing by removing it from the PARI TREK Compressor and hanging it up with the "ends down" to allow the moisture or condensation to drain out and evaporate. Use a clean damp cloth to wipe the exterior of the Tubing.
3. Disassemble the Reusable Nebulizer parts as illustrated on page 3.
4. Wash all Reusable Nebulizer parts with warm water and liquid dish soap. **DO NOT WASH THE TUBING.**
5. Rinse the Reusable Nebulizer parts thoroughly with warm water and shake out water.
6. Air dry or hand dry Reusable Nebulizer parts on a clean, lint free cloth. Reassemble Nebulizer when parts are dry and store.
7. You can also wash the Reusable Nebulizer parts in the top shelf of a dishwasher. Masks can not be washed safely in a dishwasher. **DO NOT WASH THE TUBING.** Place the Reusable Nebulizer parts in a dishwasher basket and place the basket on the top rack of the dishwasher. Remove and dry parts when the dishwasher cycle is complete.

DISINFECTING AND STERILIZING THE PARI REUSABLE NEBULIZER

1. Clean the Reusable Nebulizer as described in **Cleaning the Nebulizer** section on page 21. Every other treatment day, soak the Reusable Nebulizer parts in a solution of 1 part distilled white vinegar and 3 parts hot tap water for 1 hour. **DO NOT SOAK THE TUBING.** You can substitute commercially available respiratory equipment disinfectant (i.e., Control III®) for distilled white vinegar by following manufacturer's instructions for mixing.
2. Rinse all parts of the Reusable Nebulizer thoroughly with warm tap water and dry with a clean, lint-free cloth. Discard vinegar solution when disinfecting is complete.
3. Reusable Nebulizer parts (except Tubing and Mask) may also be sterilized by boiling them in a clean pot of water for a full 10 minutes. Hospitals and clinics can sterilize the Reusable Nebulizer parts (except Tubing and Mask) by autoclaving (a maximum of 277°F or 126°C) and following autoclave manufacturer's instructions. Dry Reusable Nebulizer parts on a clean, lint-free cloth.

CONTROL III® IS A REGISTERED TRADE MARK OF MARIL PRODUCTS, INC.

THE PARI REUSABLE NEBULIZER INCLUDES A 6 MONTH LIMITED WARRANTY FOR HOME CARE USE.

9. RECHARGEABLE BATTERY

CHARGING THE BATTERY (*optional accessory*)

1. The PARI TREK Battery must be charged for 16 hours before using it the first time, even if the Battery Level Indicator is illuminating green. Refer to PARI TREK Battery Illustration on page 3 item 2.
2. To charge the Battery, plug the Multi-Voltage AC Adapter into the wall outlet. Plug the other end of the Multi-Voltage AC Adapter into the Charge Receptacle on the top of the Rechargeable Battery. Refer to page 6 figure L. The Charge Indicator will illuminate orange indicating the Battery is charging.
3. When the Rechargeable Battery is fully charged, the Charge Indicator on the Battery will illuminate green. Unplug the Multi-Voltage AC Adapter from the Charge Receptacle on the top of the Rechargeable Battery. The amount of time it takes to recharge the Battery will depend on the age and charge level of the Battery.

NOTE: Do not leave the Multi-Voltage AC Adapter plugged into the Rechargeable Battery for more than 24 hours. The Rechargeable Battery electrically disconnects from the Multi-Voltage AC Adapter after 16 hours of charging. If left plugged in for extended periods of time, the Rechargeable Battery will slowly discharge.

4. For maximum battery performance, the Battery should be periodically charged and discharged through several cycles. In time, the battery capacity will decrease and the number of treatments per charge will be reduced.
5. When not in use, the Rechargeable Battery should be charged overnight every 1-2 months to optimize and preserve the life of the Rechargeable Battery.
6. Normal battery life is approximately 250 charge cycles. When the orange and green indicators blink alternately, the Battery has reached the end of its useful life (approximately 250 cycles) and must be replaced with a new PARI TREK Rechargeable Battery.

STORING THE RECHARGEABLE BATTERY (optional accessory)

1. The Battery should be stored FULLY CHARGED in a cool dry place.
2. Every 1-2 months, charge and discharge the Rechargeable Battery to maintain the maximum battery capacity.
3. It may take up to 5 charge/discharge cycles to return the Battery to maximum capacity after long term storage.

NOTE: If the Battery goes for a long period of time without charging, Battery damage may occur.


10. PARI TREK COMPRESSOR

MAINTENANCE

AIR FILTER: The Air Filter on the front panel of the PARI TREK Compressor is bright white when new. In time, the Air Filter will turn grey, which means it is time to replace it. You must replace the filter immediately if it gets wet or becomes clogged. The Air Filter lasts up to 500 hours of use before replacement. A good general rule is to replace the Air Filter whenever you replace your PARI Reusable Nebulizer (every 6 months). To remove the Air Filter, make sure the PARI TREK Compressor is turned off. Use a coin to turn the Air Filter cover counter clockwise and pull the cap and filter out and remove the filter from the cap. Refer to page 6 figure M. To replace filter, press the new filter into the cap with the open end visible. Place the cap and filter back into the front of the compressor and using a coin turn the filter cover clockwise to tighten as illustrated on page 6 figure M.

NOTE: Do not attempt to wash and reuse Air Filters. Do not use substitute filter material. Compressor damage may result.


CLEANING THE PARI TREK COMPRESSOR HOUSING: Use a clean, damp cloth to wipe the exterior of the PARI TREK Compressor housing. Be careful not to get liquids onto the Power Receptacle or the Battery Contacts since damage could occur.

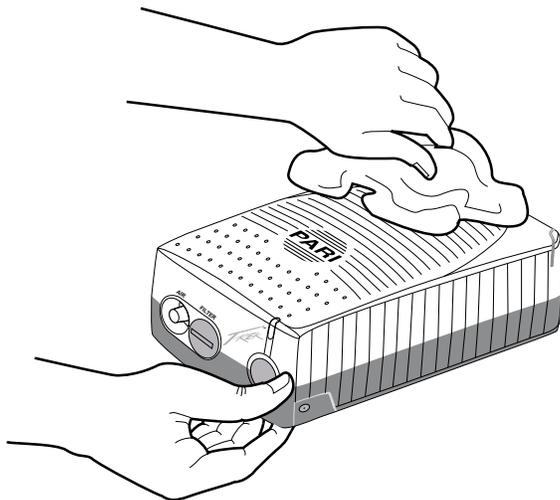

MAINTENANCE AND REPAIR: Never operate the PARI TREK Compressor if it is improperly or incompletely assembled, or damaged. Use only original PARI parts, accessories, tubing and attachments authorized by PARI Service Center for the PARI TREK Compressor Nebulizer System. Use of other parts, accessories, tubing and attachments may damage the PARI TREK Compressor unit or cause improper treatment. Do not open or disassemble the PARI TREK Compressor unit at any time. This action will void the warranty. Repairs, service and maintenance should be carried out only by an authorized PARI Service Center.

NOTE: There are no user serviceable parts inside the PARI TREK Compressor or Battery housing. Service by anyone other than an authorized PARI Service Center voids the warranty.

CLEANING THE RECHARGEABLE BATTERY: Use a clean, dry cloth to wipe the exterior of battery. Be careful not to get liquids onto the Charge Receptacle or the Battery Contacts since damage could occur.

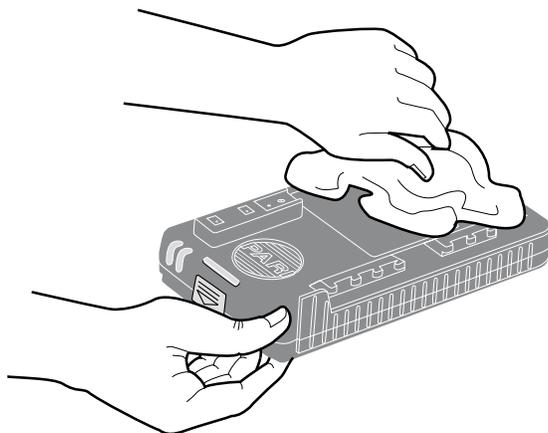

PARI TREK COMPRESSOR NEBULIZER SYSTEM SPECIFICATIONS

Weight: (Compressor only).....0.9 pound (400 grams)
With Rechargeable Battery:.....1.6 pound (740 grams)
Dimensions: (Compressor only).....5" x 3.5" x 1.75"
Electrical Requirements:.....100VAC to 240VAC
47Hz to 63Hz
Power Wattage:.....15 watts under normal load
Certification:.....IEC 60601-1-2/EN60601-1-2

Ambient operation

Temperature Range:.....50°F to 104°F (10°C to 40°C)

Transport/Storage

Ambient Temperature:.....-13°F to 158°F (-25°C to 70°C)
Humidity:.....10% to 95%

Optional Rechargeable Battery:

Operating time with battery:.....Approximately 60 minutes
Number of treatments per charge:.....Up to 8 treatments when
nebulizing 2.5ml of medication
using a PARI Reusable Nebulizer
Normal time to charge:.....Approximately 5 Hours

PARI TREK COMPRESSOR LIMITED WARRANTY

PARI offers a limited three (3) year warranty on the PARI TREK Compressor and six (6) months on accessories. PARI warrants the PARI TREK Compressor and its parts will be free from defects in workmanship and materials for a period of three (3) years from the date of original retail purchase of the compressor and six (6) months on the PARI Reusable Nebulizer, Battery accessories. This warranty does not cover failures of the nebulizer or tubing. This warranty does not extend to failures resulting from accident, misuse, abuse, alteration, use of unauthorized service, parts or cleaning solutions, or failure to comply with **Instructions For Use**.

If the warranted equipment should fail during the warranty period, PARI, at its option, will repair or replace parts at issue, provided the claim is bona fide. It shall be the responsibility of the purchaser to pack and return the equipment in a manner to avoid shipping damage. The shipment must be prepaid to PARI Respiratory Equipment, Inc. This warranty covers the cost of labor incurred in the removal or replacement of the warranted component parts at authorized PARI Service Centers only. To make a claim under this warranty, notify the authorized service center by calling PARI Patient Connection toll free (USA) at 1.877.943.3931, then press 1.

THERE ARE NO OTHER EXPRESS WARRANTIES. ALL IMPLIED WARRANTIES ON THE PARI TREK AND ITS PARTS, INCLUDING IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED TO THIRTY-SIX MONTHS FROM THE DATE OF FIRST RETAIL PURCHASE. ALL EXPRESS OR IMPLIED WARRANTIES ON ANY OTHER PARTS, INCLUDING THE IMPLIED WARRANTIES ON ANY OTHER PARTS OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE DISCLAIMED. Some states do not allow limitations on how long an implied warranty lasts, so the above limitations may not apply to you.

The sole and exclusive remedy under this warranty is to repair or replace the PARI TREK and/or its parts at the option of PARI. IN NO EVENT SHALL PARI BE LIABLE FOR INCIDENTAL OR CONSEQUENTIAL DAMAGES TO THE PURCHASER OR ANY THIRD PARTY, INCLUDING WITHOUT LIMITATION, DAMAGES ARISING FROM LOST PROFITS, LOSS OF BUSINESS OPPORTUNITY, LOSS OF PROPERTY, ECONOMIC LOSSES, OR STATUTORY OR EXEMPLARY DAMAGES WHETHER IN NEGLIGENCE, WARRANTY, STRICT LIABILITY OR OTHERWISE. Some states do not allow the exclusion of incidental or consequential damages, so the above limitations or exclusions may not apply to you. This warranty gives you specific rights. You may also have other rights, which vary State to State.

SERVICE AND REPAIR

If you should have any questions about your PARI TREK Compressor Nebulizer System:

1. Please contact PARI Patient Connection toll free (USA) at 1.877.943.3931, then press 1 for the authorized service center.
2. In writing, describe the malfunction or symptoms of the PARI TREK Compressor. Include your full name, address, and daytime telephone number. For the PARI Nebulizer clean, disinfect, and/or sterilize prior to returning. Dry the nebulizer and place in a plastic bag.
3. Send the written description and the PARI TREK Compressor in a protective box to the address provided by PARI Patient Connection.

PATENTS PENDING

46D0001 Rev 9/02



Specialist in effective inhalation

PARI RESPIRATORY EQUIPMENT, INC.
3 Justin Court, Monterey, California 93940
Toll Free (USA) 1.877.943.3931 Fax 1.800.727.4112

www.**PARI**.com
E-mail ProductInfo@PARI.com
Respiratory CEU programs available
© 2002 PARI Respiratory Equipment, Inc.

