Compressor Nebulizer System

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CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician or licensed healthcare professional.

Contents: (1) Inspiration Elite Compressor with Instructions
(1) Nebulizer with Tubing

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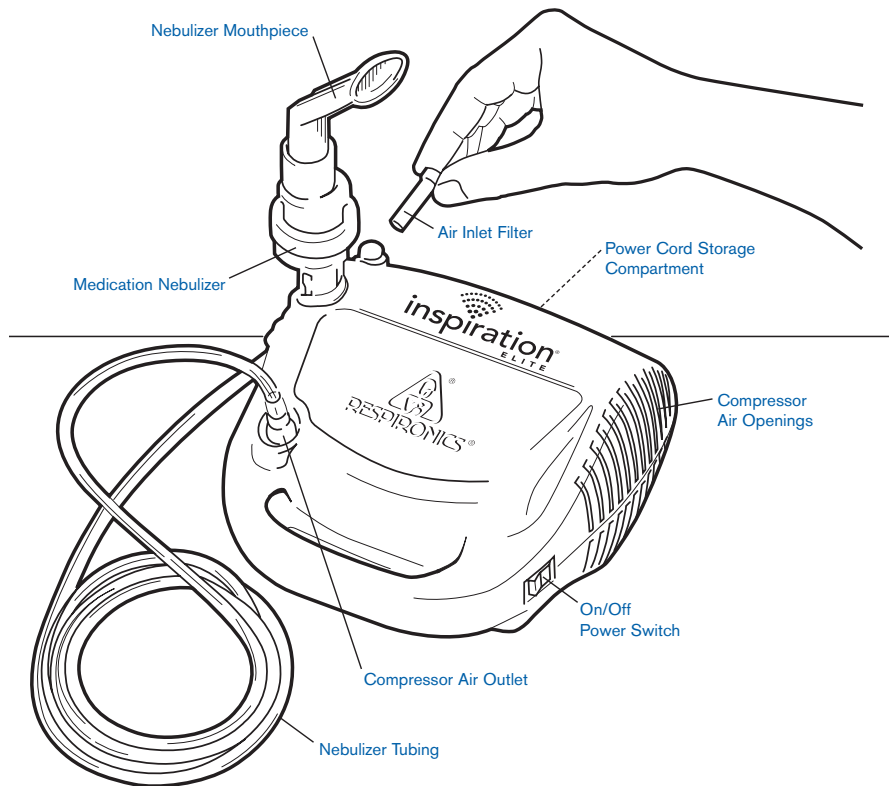
Sin látex

inspiration[®]
ELITE

Inspiration® Elite is an AC-powered air compressor nebulizer system intended to provide a source of compressed air for medical purposes for use in home healthcare. It is to be used with a pneumatic nebulizer to produce aerosol particles of medication for respiratory therapy for both children and adults.

Unpacking Your Inspiration Elite System

When you unpack your Inspiration Elite from its shipping case:

- Carefully examine the contents.
- Save the shipping carton.
- Make sure you have all the content items listed and that they are not damaged.
- Report anything missing or damaged to Respironics.

Do not install, maintain or operate this equipment without first reading and understanding these instructions for use. If you are unable to understand any portion of these instructions, contact your healthcare provider or call Respironics Customer Service at 1-800-345-6443 or 1-724-387-4000.

IMPORTANT SAFEGUARDS

When using electrical products, especially when children are present, safety precautions should always be followed, including the following:

READ ALL INSTRUCTIONS BEFORE USING

⚠ DANGER – To reduce the risk of electrocution:

- Always unplug this product immediately after using.
- Do not place or store product where it can fall or be pulled into a tub or sink.
- Do not place in or drop into water or other liquid.
- Do not reach for product that has fallen into water. Unplug immediately.

⚠ WARNING – To reduce the risk of burns, electrocution, fire or injury to persons:

- A product should never be left unattended when plugged in.
- Close supervision is necessary when this product is used by, on, or near children or physically challenged individuals.
- Use this product only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.
- Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the product to a service center for examination and repair.
- Keep the cord away from heated surfaces.

- Never block the air opening of this product or place it on a soft surface, such as a bed or couch, where the air openings may be blocked. Keep the air openings free of lint, hair, and the like.
- Never use while sleeping or drowsy.
- Never drop or insert any object into any openings.
- Do not use outdoors or operate where aerosol (spray) products are being used or where oxygen is being administered in a closed area such as an oxygen tent.
- Do not use compressor and nebulizer separately.

SAVE THESE INSTRUCTIONS

How To Use Your Inspiration Elite

1. Check the medication nebulizer cup for damage. (The nebulizer cup enclosed with your Inspiration Elite is disposable and should be replaced regularly. Ask your healthcare provider for replacement guidelines and supplies.)
2. Place your Inspiration Elite unit on a firm, flat surface near an electrical outlet and in an area where you can be seated comfortably during your treatment.
3. Plug the power cord into the electrical outlet.
4. Do not operate the unit without the filter or with a clogged (dirty) filter. Operating the unit with a dirty filter can impair proper operation and cause damage that is not covered by the warranty.
5. Wash your hands before preparing your medication.
6. Assemble your nebulizer cup and add your prescribed medicine following your prescription and the separate nebulizer instructions.
7. Connect one end of the tubing to the bottom of the cup and the other end of the tubing to the air outlet on the unit.
8. Turn the power switch on (**I**) to begin your treatment.
9. When your treatment is complete, turn the power switch off (**O**) and unplug the unit.
10. Clean the medication nebulizer cup according to its instructions.

Care and Cleaning of Your Inspiration Elite

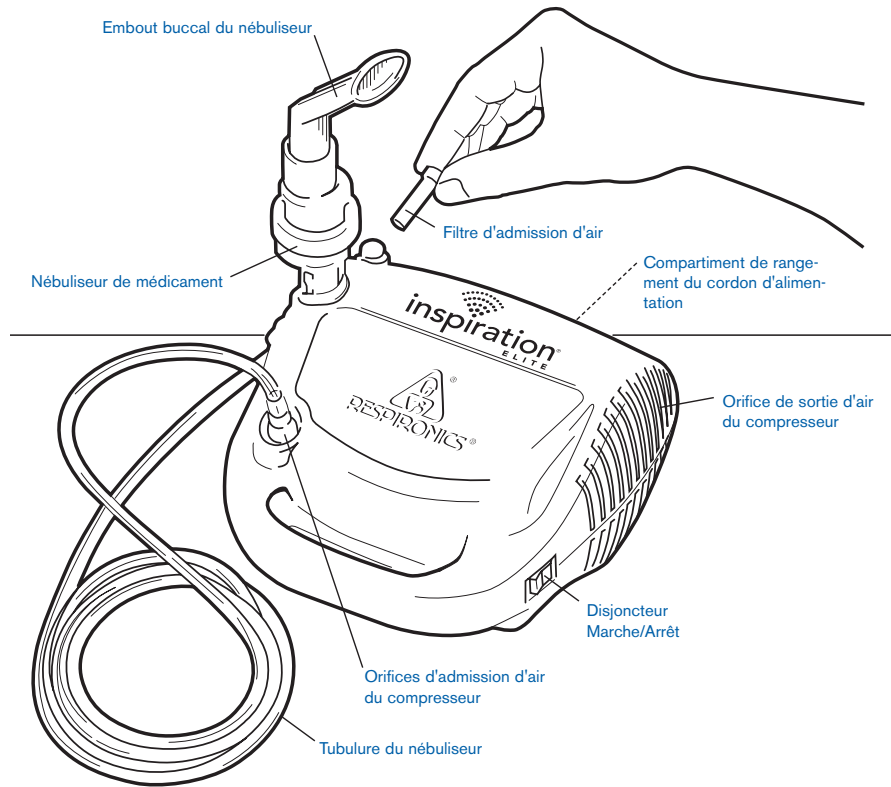
CAUTION: To reduce risk of electric shock, DO NOT DISASSEMBLE

1. Unplug the unit before cleaning.
2. Use a damp cloth to wipe down the outside of the compressor at least once each month (or more if necessary).

3. Check the filter at least once each week. Replace the filter when dirty. Replacement filters are available where you purchased this product (Package of 4: Part No. 1022120 or Package of 50: Part No. 1022121).
4. To change the filter:
 - Lift up the filter and remove it by twisting and pulling it straight up.
 - The filter should be replaced when it becomes discolored. Do not wash or reuse the filter.
5. After each cleaning, check that the unit is working properly by verifying that air still comes out of the nozzle when the unit is turned to the "on" position.
6. The unit is oil-free. Do not lubricate.
7. Service should only be performed by an authorized dealer. Unauthorized repair voids warranty.

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Standard Specifications:

Electrical Requirements:	115VAC~ 60Hz, 2.0 Amps
Power Wattage:	90 watts @ 10 psig (approx.)
Thermal Overload Protector:	Thermal – Fuse UL2111 Class B
Performance:	Average Flow Rate: 6.4—7.6 LPM@10psig Maximum Pressure: 25 psig (Self-Resetting Pressure Relief Valve)
Replacement Intake Filter:	Respironics Part No. 1022120 or 1022121
Size:	7.5" x 7.5" x 4.2"
Weight:	3.3 lbs.
Regulatory Listing:	Listing by Underwriters Laboratories Inc. covers this air pump only. It does not cover any accessory equipment.

Operating Temperature:	50°F - 104°F (10°C - 40°C)
Operating Humidity:	RH 10% RH to 95% RH, non-condensing
Storage/Transport Temperature:	-13°F - 140°F (-25°C - 60°C)
Storage/Transport Humidity:	RH 10% RH to 95% RH, non-condensing
Storage/Transport Atmospheric Pressure:	P 690 – 1060 hPa

Symbol Glossary:

△ Attention: Read accompanying documents
 ○ OFF (power)
 SN Serial Number

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Warranty

Respironics, Inc. warrants the Inspiration Elite compressor to be free from defects in materials and workmanship under normal use and operation for a period of 5 years from date of purchase from Respironics, Inc. The warranty is limited to repair or replacement, at Respironics, Inc.'s sole option, of any such component or equipment claimed to be defective when claim is shown to be bona fide by evaluation by Respironics, Inc. This warranty does not extend to any components or equipment subjected to misuse, improper operation, accidental damage, or unauthorized repairs, and is not extended to charges of, or for, labor repairs. All items returned must be properly packaged and shipped, prepaid, by the distributor servicing the unit. Respironics, Inc. shall not be liable to purchaser or others for loss of use of equipment or for indirect, or incidental or consequential damages that might arise.

Made in China by Rietschle Thomas for Respironics New Jersey, Inc. Inspiration is a registered trademark of Respironics, Inc. ©2004 Respironics, Inc. and its affiliates. All rights reserved.

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Respironics New Jersey, Inc.
41 Canfield Road
Cedar Grove, NJ 07009-1201

1020717 REV1