

ACCESSING AND CHANGE NPB290 DEFAULTS SETS

The menu can be accessed by pressing
↑UPPER AND LOWER↓
alarm buttons simultaneously for 3 seconds.
Menu options have been accessed when a “1” appears
in the pulse rate display.

Push the Adjust up bottom on the right and select “9”
to save the new values as the current power-on default
settings, press the Upper Alarm limits button. Three
tones will sound to indicate that the changes have been
accepted.

***please note change the alarms settings before
doing*** this and the patient cable must be removed